
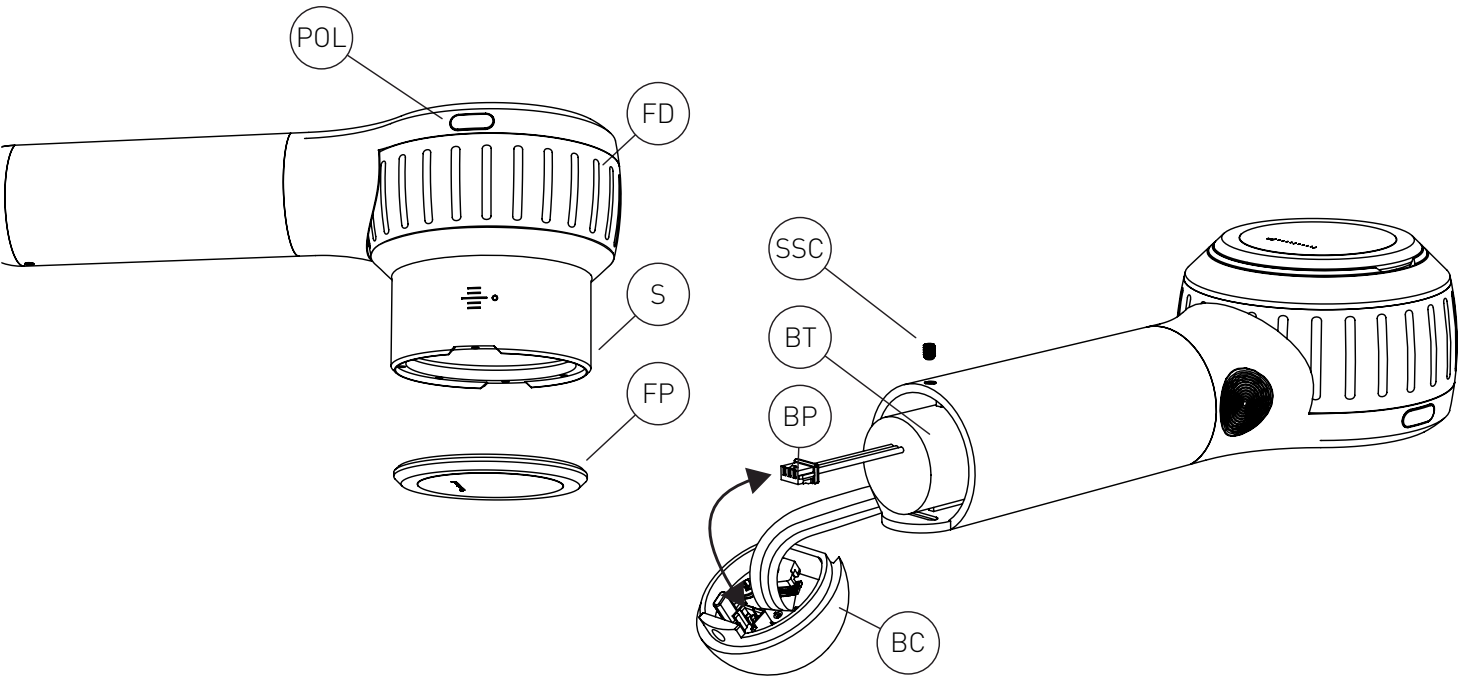
dermLite[®] DL4

English	3
Bosanski	4
Čeština	5
Dansk	6
Deutsch	7
Eestlane	8
Español	9
Français	10
Gaeilge	11
Hrvatski	12
Italiano	13
Latvietis	14
Lietuvis	15
Magyar	16
Nederlands	17
Polskie	18
Português	19
Română	20
Slovenščina	21
Slovenský	22
Suomi	23
Svenska	24
Türk	25
Ελληνικά	26
български	27
русский	28
Український	29
한국어	30
日本語	31
简体中文	32

dermLite DL4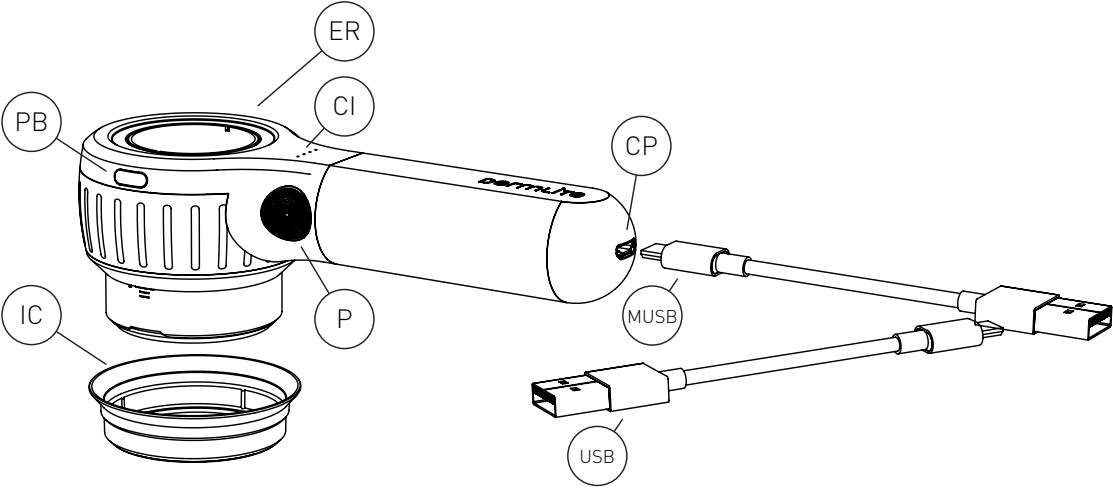

Instructions

Intended Use

This device is intended for medical purposes to illuminate body surfaces. It is used for non-invasive visual examination of intact skin.

This battery-operated product is designed for external examination only within professional healthcare facilities by medical professionals. Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

CAUTION: Do not look directly into the LED light. Patients must close their eyes during examinations.

In case of a serious incident with the use of this device, notify DermLite immediately and, if required by local regulations, your national health authority.

CAUTION: Do not use the device in fire or explosive risk area (e.g. oxygen-rich environment).

WARNING: This product can expose you to chemicals including methylene chloride and hexavalent chromium, which are known to the State of California to cause cancer or reproductive toxicity. For more information go to www.P65Warnings.ca.gov.

Electromagnetic Compatibility

This device complies with the EMC Emissions and Immunity level requirements of the standard IEC 60601-1-2:2014. The emission characteristics of this equipment make it suitable for use in professional healthcare environment as well as residential environment (CISPR 11 Class B). This equipment offers adequate protection to radio communication service. In the rare event of interference to the radio communication service, the user might need to take mitigation measures, such as relocating or re-orienting equipment.

WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

WARNING: Use of accessories other than those provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING: Portable RF transmitters should be used no closer than 30 cm (12 inches) to any part of the device. Otherwise, degradation of the performance of this equipment could result.

IMPORTANT: Before initial use, please charge the internal lithium ion battery. See "Battery and Charging" for details.

DermLite DL4 is a smartphone-compatible pocket dermatoscope designed to view skin lesions using cross-polarized or non-polarized light and high magnification and clarity. A high-quality 10x lens with superb color correction and reduced image distortion produces an image rich in surface detail.

IceCap®

To support your infection control measures,

snap one of the included disposable IceCaps (IC) over the installed faceplate (FP). To remove and dispose, simply pull it off.

Hold your DermLite DL4, with the LEDs aimed in the direction of the lesion to be examined, approximately 25mm (1") above the skin. Push the power button (P) to turn on the device or push the two mode buttons (POL & PB), located at the head, simultaneously. Look through the lens and move the device closer or farther away from the skin to focus the image. To toggle between non-polarized mode for immersion fluid dermoscopy and cross-polarized illumination, briefly push any button. Push and hold (POL) for about 1 second to toggle between two brightness levels. For dermoscopy with skin contact, rotate the Focus Dial (FD) to extend the spacer (S) and focus the image. To turn off your DermLite, push the power button (P) for about 1 sec. or push the two mode buttons (POL & PB), located at the head, simultaneously. To save power, the unit automatically turns off after 3 minutes.

Photography

This DermLite may be connected with mobile devices or cameras using DermLite magnetic adapters (sold separately) which attach to the extension ring (ER). Prior to taking images, wipe the faceplate (FP) clean and extend the spacer (S) to the "0" position.

Removing the Faceplate

The faceplate (FP) may be pulled off the Spacer (S) by grabbing it at the fingernail notches.

PigmentBoost®

Press (PB) briefly to activate PigmentBoost lighting in addition to white light. To enter the exclusively orange PigmentBoost Plus mode, press the button (PB) for about 1 second; briefly press (PB) or (POL) to exit.

Battery and Charging

CAUTION: This device uses a special 3.7V 1400 mAh lithium ion battery, which can only be purchased from DermLite or from an authorized DermLite dealer. Do not under any circumstances use another battery other than the one designed for this unit.

This device is equipped with a four-level Charge Indicator (CI). When the unit is turned on and fully charged, all four LEDs light up. When less than 75% of battery life remains, three LEDs are illuminated. If more than half the battery is drained, two LEDs are on, while a single LED indicates that less than 25% of battery capacity remains. To charge your device, connect the supplied USB-C to USB cable to the Charging Port (CP) and any IEC 60950-1 compliant (5V) USB port. The Charge Indicator (CI) pulses to indicate charging, with the number of pulsing LEDs corresponding with charge level. When charging is complete, all four indicator LEDs illuminate. After years of use, you may wish to re-place the battery, which is only available directly from DermLite or an authorized DermLite dealer.

After hundreds of charges, the battery's

capacity will be reduced to the point where it may be more convenient to replace the battery. To remove the battery (BT), remove the battery cap (BC) by unscrewing the set screw (SSC) using the wrench supplied with each DL4 replacement battery. Disconnect the small white battery plug (BP) and slide the battery (BT) out. Insert a new battery and reconnect the battery plug. Replace the battery cap (BC) and secure it with the set screw (SSC).

Troubleshooting

Please check www.derm-lite.com for the most current troubleshooting information. If your device requires servicing, visit www.derm-lite.com/service or contact your local DermLite dealer.

Care and Maintenance

CAUTION: No modification of this equipment is allowed.

Your device is designed for trouble-free operation. Repairs shall be made only by qualified service personnel.

Cleaning

The exterior of your device, except the optical parts, may be wiped clean with isopropyl alcohol (70% vol.) prior to use on a patient. The lens should be treated as high-quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

Warranty: 10 years for parts and labor. The battery is warranted for 1 year.

Disposal

This device contains electronics and a lithium-ion battery that must be separated for disposal and may not be disposed of with general household waste. Please observe local disposal regulations.

This set includes:

DermLite (24 LEDs, 30 mm lens system with 10x magnification, cross-polarized and non-polarized illumination, retractable spacer, removable faceplate with 10 mm reticle), silicone sleeve, belt case, charging cable, 5 IceCaps.

Technical Description

Visit www.derm-lite.com/technical or contact your local DermLite dealer.

Manufacturer
Proizvođač
Výrobce
Fabrikant
Hersteller
Tootja
Fabricante
Fabricant
Déantúsóir
Proizvođač
Fabbrikante
Ražotājs
Gamintojas
Gyártó
Fabrikant
Producent
Fabricante
Producător
Proizvajalec
Výrobca
Valmistaja
Tillverkare
Üretici firma
Κατασκευαστής
Производитель
производитель
Виробник
제조사
メーカー
製造商

Catalog Number
Kataloški broj
Katalogové číslo
Katalognummer
Katalognummer
Katalooginumber
Numero de catalogo
Numéro de catalogue
Uimhir Catalog
Kataloški broj
Numero di catalogo
Kataloga numurs
Katalogo numeris
Katalógus szám
Catalogus nummer
Numer katalogu
Número de catálogo
Catalog Number
Kataloška številka
Katalogové číslo
Luettelonumero
Katalognummer
Katalog numarası
Αριθμός καταλόγου
Каталожен номер
Номер каталога
Номер каталогу
카탈로그 번호
カタログ番号
目録番号

Serial Number
Serijski broj
Sériové číslo
Seriennummer
Seriennummer
Seerianumber
Número de serie
Numéro de série
Uimhir Sraithuimhir
Serijski broj
Numero di serie
Sērijas numurs
Serijos numeris
Sorozatszám
Seriennummer
Numer seryjny
Número de série
Număr de serie
Serijska številka
Sériové číslo
Sarjanumero
Seriennummer
Seri numarası
Σειριακός αριθμός
Серийный номер
Серийный номер
Серійний номер
일련 번호
シリアルナンバー
序列号

Date of Manufacture
Datum proizvodnje
Datum výroby
Fremstilling dato
Hersteldatum
Tootmiskuupäev
Fecha de manufactura
Date de fabrication
Dáta an Déantúsaíochta
Datum proizvodnje
Data di produzione
Izgatavošanas datums
Pagaminimo data
Gyártás dátuma
Fabricagedatum
Data produkcji
Data de fabricaço
Data fabricatiei
Datum izdelave
Dátum výroby
Valmistuspäivämäärä
Tillverkningsdatum
Üretim tarihi
Ημερομηνία παραγωγής
Дата на производство
Дата производства
Дата виготовлення
제조 일자
製造日
生产日期

Follow instructions for use
Pratite uputstva za upotrebu
Postupujte podle pokynů k použití
Følg brugsanvisningen
Gebruichsanweisung befolgen
Järgige kasutusjuhendit
Siga las instrucciones de uso.
Suivez les instructions d'utilisation
Lean na treoracha úsáide
Slijedite upute za uporabu
Seguire le istruzioni per l'uso
Izpildiet lietošanas instrukcijas
Laikykites naudojimo instrukcijų
Kövesse a használati útmutatót
Volg de instructies voor gebruik
Postępuj zgodnie z instrukcjami użytkowania
Siga as instruções de uso
Urmați instrucțiunile de utilizare
Upoštečajte navodila za uporabo
Postupujte podľa pokynov na použitie
Seuraa käyttöohjeita
Följ instruktionerna för användning
Kullanım talimatlarını izleyin
Ακολουθήστε τις οδηγίες χρήσης
Следвайте инструкциите за употреба
Следуйте инструкциям по применению
Дотримуйтеся інструкцій із використання
사용 지침을 따르십시오
使用説明書に従ってください
遵循使用说明

Medical Device
Medicinski savjet
Zdravotnické zařizení
Medicinsk udstyr
Medizinisches Gerät
Meditsiiniline seade
Dispositivo médico
Dispositif médical
Feiste leighis
Medicinski uređaj
Dispositivo medico
Medicīniska iekārta
Medicininis prietaisas
Orvosi eszköz
Medisch apparaat
Urządzenie medyczne
Aparelho médico
Aparat medical
Medicinski pripomoček
Zdravotnicke pomôcky
Lääketieteellinen laite
Medicinsk utrustning
Tibbi cihaz
Ιατρική συσκευή
Медицинско изделие
Медицинский прибор
Медицинний прилад
의학 기기
医療機器
医疗设备

Direct Current
Prava struja
Stejnoseměrný proud
Jævnstrøm
Gleichstrom
Alalisvool
Corriente continua
Courant continu
Sruth díreach
Prava struja
Corrente continua
Līdzstrāva
Nuolatinė srovė
Egyenáram
Gelijkstroom
Prąd stały
Corrente direta
Curent continuu
Enosmerni tok
Priamy prúd
Tasavirta
Likström
Doğru akım
Συνεχές ρεύμα
Постоянен ток
Постоянный ток
Постійний струм
직류
直流
直流电

Do not dispose as unsorted municipal waste
Ne odlagati kao nesortirani komunalni otpad
Nevyhazujte do netříděného komunálního odpadu
Bortskaf ikke som usorteret kommunalt affald
Nicht als unsortierter Hausmüll entsorgen
Ärge visake sortimata olmejäätmete hulka
No desechar como basura municipal sin clasificar
Ne pas jeter comme déchet municipal non trié
Ná déan diúscairt mar dhramhaíl chathrach neamhshaoifa
Ne odlagati kao nesortirani komunalni otpad
Non gettare tra i rifiuti urbani non differenziati
Neatbrīvojieties no nešķirotiem sadzīves atkritumiem
Neišmeskite kaip nerūšiuotos komunalinės atliekos
Ne dobja el válogatás nélkülű kommunális hulladéként
Gooi ze niet weg als ongesorteerd gemeentelijk afval
Nie wyrzucaj razem z niesortowanymi odpadami komunalnymi
Não descarte como lixo municipal não classificado
Nu aruncați ca deșeuri municipale nesortate
Ne odlagajte med nesortirane komunalne odpadke
Nevyhadzujte do netriedeného komunálneho odpadu
Älä hävitä lajittelemattoman yhdyskuntajätteenä
Kassera inte som osorterat kommunalt avfall
Ayrılmamış belediye atığı olarak atmayın
Μην απορρίπτετε ως αστικά απόβλητα
Не изхвърляйте като несортирани битови отпадъци
Не выбрасывать в качестве несортированных бытовых отходов.
Не викидайте як несортовані побутові відходи
분류되지 않은 도시 폐기물로 처리하지 마십시오
分別されていない一般廃棄物として処分しないでください
请勿作为未分类的城市垃圾处理

Indicates the authorised representative in Switzerland
Ovlašteni zastupnik u Švicarskoj
Autorizovaný zástupce ve Švýcarsku
Autoriseret repræsentant i Schweiz
Bevollmächtigter Vertreter in der Schweiz
Volitatus esindaja Šveitsis
Representante autorizado en Suiza
Mandataire en Suisse
Ionadaí údaraithe san Eilvéis
Ovlašteni predstavnik u Švicarskoj
Rappresentante autorizzato in Svizzera
Pilinvarotais pārstāvis Šveicē
Igalīotais atstovas Šveicarijā
Meghatalmazott képviselő Svájcban
Gemachtigde vertegenwoordiger in Zwitserland
Autoryzowany przedstawiciel w Szwajcarii
Representante autorizado na Suíça
Reprezentant autorizat în Elveția
Pooblaščen zastopnik v Švici
Autorizovaný zástupca vo Švajčiarsku
Valtuutettu edustaja Sveitsissä
Auktoriserad representant i Schweiz
İsviçre'deki yetkili temsilci
Εξουσιοδοτημένος αντιπρόσωπος στην Ελλάδα
Оторизирован представитель в Швейцария
Уполномоченный представитель в Швейцарии
Уповноважений представник у Швейцарії
스위스 공인 대리인
スイスの正規代理店
在瑞士的授权代表

European authorized representative
Europski ovlašteni predstavnik
Evropský zplnomocněný zástupce
Europæisk autoriseret repræsentant
Europäischer Bevollmächtigter
Euroopa volitatus esindaja
Representante autorizado europeo
Représentant autorisé européen
Ionadaí údaraithe Eorpach
Europski ovlašteni predstavnik
Rappresentante autorizzato europeo
Eiropas pilnvarotais pārstāvis
Europos įgaliotasis atstovas
Európai meghatalmazott képviselő
Europese gemachtigde vertegenwoordiger
Europejski upoważniony przedstawiciel
Representante autorizado europeu
Reprezentant autorizat european
Evropski pooblaščen zastopnik
Európsky splnomocnený zástupca
Eurooppalainen valtuutettu edustaja
Europeiska auktoriserade representant
Avrupa yetkili temsilcisi
Ευρωπαϊός εξουσιοδοτημένος εκπ
Европейски упълномощен представител
Европейский уполномоченный представитель
Европейский уполномочений представитель
Ευρωπαϊκή εξουσιοδοτημένος εκπ
유럽 공인 담당자
歐洲公認代表
歐洲授權代表

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Responsible Person Limited
Summit House, 4-5 Mitchell Street
Edinburgh, EH6 7BD Scotland

CompLife Swiss s.a.
Via Luganetto 4
6962 Viganella (TI)

Patents/Trademarks: www.dermlite.com/patents

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DermLite LLC 31521 Rancho Viejo Rd Ste 104
E. C. REP Ltd